


B.E.S.T. Practice
BIOPATCH®
Every Single Time.*

Fight CRBSIs! Use BIOPATCH® Every Single Time.

How to apply BioPATCH® Disk:

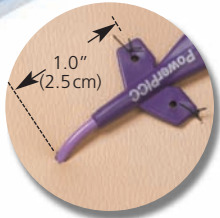

1 Prepare the insertion site. Secure catheter at least 1" (2.5 cm) from insertion site.



2 Ensure skin prep is completely dry. Place BIOPATCH® around catheter **printed side up**.



3 Align catheter with radial slit. Ensure edges of slit touch.



4 Secure catheter and BIOPATCH® with transparent film dressing. Ensure complete contact between BIOPATCH® and skin.

How to remove BioPATCH®:




1 Peel film away from catheter.



2 BIOPATCH® comes away with film.

Do's and Don'ts:

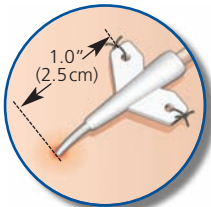

Do secure catheter at least 1" (2.5 cm) from insertion site. This allows proper placement of BIOPATCH®.




Don't place white side up. Antimicrobial white side must face skin. If wrong, change immediately.



Do align radial slit with catheter. This helps easy removal.



Don't allow slit edges to straddle catheter. Edges of slit must touch to assure efficacy.

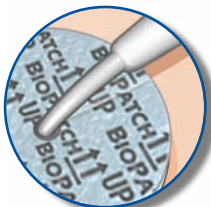

Do ensure edges of slit touch. This assures efficacy.



Don't secure catheter too close to entry point. This will prevent proper placement of BIOPATCH®.

Ordering BIOPATCH®



ORDER CODE	4150	4151	4152
SIZE	1" disk (2.5cm) w/4.0mm center hole	3/4" disk (1.9cm) w/1.5mm center hole	1" disk (2.5cm) w/7.0mm center hole



Don't place BIOPATCH® on catheter. BIOPATCH® must have complete contact with skin to assure efficacy.

For Full Prescribing Information or technical support, call **1-877-ETHICON (1-877-384-4266)** or visit **www.BioPATCH.com**

To place an order, call **1-800-255-2500**