

PUREWICK®


FEMALE EXTERNAL CATHETER

DIRECTIONS FOR USE

Simple, non-invasive urine management for women



SETUP

1. Connect canister to wall suction and set to 40mmHg continuous suction.
2. Using standard suction tubing, connect the PUREWICK® Female External Catheter to the collection canister.

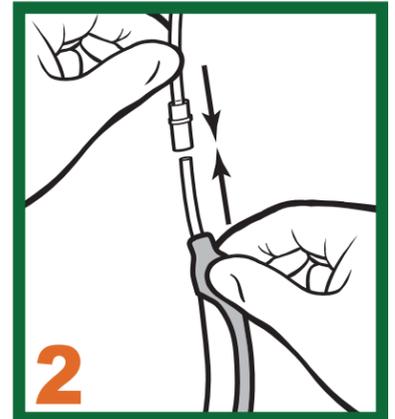


PLACEMENT

3. **A** Perform perineal care and assess skin integrity. **B** Separate legs, gluteus muscles, and labia. Palpate pubic bone as anatomical marker.
4. **A** With soft gauze side facing patient, align distal end at gluteal cleft. Gently tuck soft gauze side between separated gluteus and labia. **B** Slowly place legs back together.

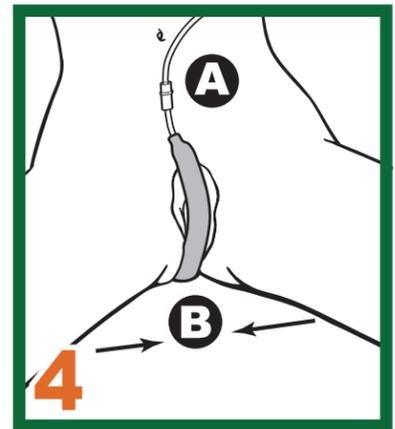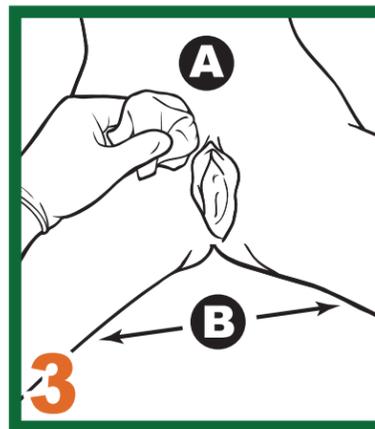

REMOVAL & MAINTENANCE

5. **A** Fully separate the legs, gluteus, and labia. **B** To avoid potential skin injury upon removal, gently pull the PUREWICK® Female External Catheter directly outward. Ensure suction is maintained while removing.
6. Replace every 8-12 hours or if soiled with feces or blood. Assess skin for compromise and perform perineal care prior to placement of a new PUREWICK® Female External Catheter.

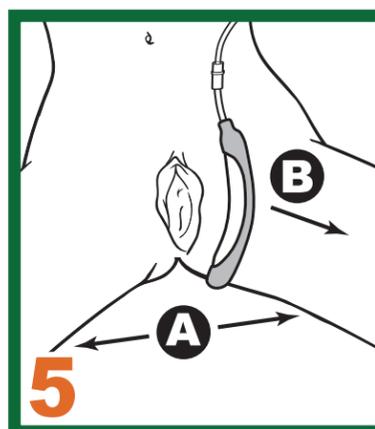

BARD | MEDICAL

Advancing Lives and the Delivery of Healthcare.™

C. R. Bard, Inc.
Covington, GA 30014
1.800.526.4455
www.bardmedical.com

Bard, Advancing Lives and the Delivery of Healthcare and PureWick are trademarks and/or registered trademarks of C. R. Bard, Inc. ©2017 C. R. Bard, Inc. All Rights Reserved. 1706-35 BMD/FECS/0817/0015

Indications: The PureWick® Female External Catheter is intended for non-invasive urine output management in female patients.

Caution: This Product Contains Dry Natural Rubber

Contraindications: The PureWick® Female External Catheter should not be used on patients with urinary retention.

Warnings: To avoid potential skin injury, never push or pull the PureWick® Female External Catheter against the skin during placement or removal. Never insert the PureWick® Female External Catheter into the vagina, anal canal, or other body cavities. Do not use the PureWick® Female External Catheter with a bedpan or any material that does not allow for sufficient airflow. Discontinue use if an allergic reaction occurs. After use, this product may be a potential biohazard. Dispose of in accordance with applicable local, state and federal laws and regulations.

Please consult package insert for more detailed safety information and instructions for use.